

# CAM360 AmnioGraft™ User Guide

## Supplies for CAM360 AmnioGraft User

12mm Sterile CAM360 AmnioGraft™ (CAM360 AG) Product

Sterile Soft Bandage Contact Lens (BCL)

Sterile Surgical Tray

Sterile Forceps

Sterile Polyester Tipped Applicator Swab



## Pressing BCL onto CAM360 AG on Surgical Tray




1. Open surgical tray, swab, contact lens, and packaging for CAM360 AG. Retrieve CAM360 AG with forceps.



2. Place CAM360 AG flat onto the surgical tray surface with the gridded side facing down.



3. Use forceps to remove paper backing, one side at a time, while using swab to press and spread CAM360 AG.

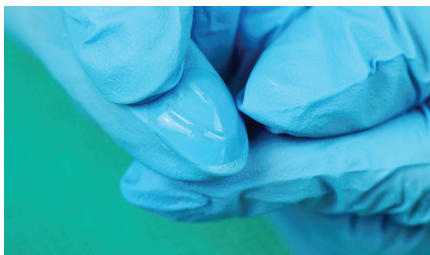

4. Retrieve and invert BCL with a gloved finger. Excess liquid should be removed.



5. Press to adhere inverted BCL onto the spread out CAM360 AG on the surgical tray.



6. Revert BCL back to its normal shape and use swab to spread CAM360 AG if there are any wrinkles/folds.

## CAM360 AG & Final Steps

7. BCL with CAM360 AG is now ready to be placed on the corneal surface using the same technique for inserting a contact lens using an index finger.

*Note: Forceps are used for demonstration purposes only but are not meant to be used for insertion.*

