

# CAM360 AmnioGraft™

## Patient Guide

Use of CAM360 AG with  
**Bandage Contact Lens  
(BCL)**



**Up to 48-hours**

### Clinical Applications



Dry Eye Disease



Neurotrophic Keratitis  
*Reduced Corneal Sensitivity*

### Care Instructions

- Ensure hands are cleaned before any contact with your eye during treatment.
- Treated eye may be taped. To remove the tape, the eye should be kept shut and the tape should be gently peeled.
- **Apply preservative-free artificial tears, every 30-minutes to 1-hour to avoid the BCL from drying out.**
- **Apply preservative-free artificial tears before going to sleep and upon waking up, to avoid the BCL from drying out.**
- **Avoid** rubbing the eye during treatment.
- **Avoid** direct water exposure to your eye during treatment.
- **Avoid** ocular exposure to aerosols, hairsprays, cosmetics, lotions, etc.



**If the treatment duration exceeds 48-hours**, your Healthcare Provider will prescribe prophylactic antibiotic eye drops. **These drops should be administered at least three times daily to the treated eye.**

### Allergies & Warnings

- Should you experience worsening pain, redness, discharge, excessive tearing, or swelling, contact your healthcare provider immediately.