

CAM360 AmnioGraft™

Patient Guide

Use of CAM360 AG with Collagen Shield (CS)



Up to **72-hours**

Clinical Applications



Dry Eye Disease



Neurotrophic Keratitis
Reduced Corneal Sensitivity

Care Instructions

- Ensure hands are cleaned before any contact with your eye throughout treatment.
- **Administer preservative-free artificial tears at least 3 times a day.**
- Treated eye should remain taped between 48 to 72-hours.
- To remove the tape, the eye should be kept shut and the tape should be gently peeled.
- After tape is removed, continue preservative-free artificial tears 3 times a day.
- If the CS or tissue falls out, or slips out after tape removal, there is no need for concern. Notify your healthcare provider at your follow-up appointment.
- **Avoid** rubbing the eye during treatment.
- **Avoid** direct water exposure to your eye during treatment.
- **Avoid** ocular exposure to aerosols, hairsprays, cosmetics, lotions, etc.

Allergies & Warnings

- If you have a known allergy to collagen or bovine products, you must inform your healthcare provider. **Do not receive CS treatment.** Potential allergic reaction symptoms may include redness, tearing, itching, or swelling.
- You may experience a temporary stinging sensation following the insertion of the CS which should shortly neutralize. If not, please advise your healthcare provider.