

CAM360 AmnioGraft™

Patient Guide

Use of CAM360 AG with Bandage Contact Lens (BCL)



Up to **48-Hours**

Clinical Application



Mild to Moderate:
Superficial Punctate Keratitis (SPK)
Dry Eye Disease (DED)



Neurotrophic Keratitis (NK)
Stage 1

Care Instructions

- Ensure hands are cleaned before any contact with your eye during treatment.
- **Apply preservative-free artificial tears every 30-minutes to 1-hour during the day, as well as before going to sleep and upon waking to keep the BCL from drying out.**
- If the BCL or tissue falls out, do not attempt to re-insert the BCL or tissue into the eye. Notify your healthcare provider at your follow-up appointment.
- **Avoid** rubbing the eye during treatment as this may dislodge the BCL or cause discomfort.
- **Avoid** direct water exposure to your eye during treatment.
- **Avoid** ocular exposure to aerosols, hairsprays, cosmetics, lotions, etc.



If the treatment duration exceeds 48-hours, your healthcare provider may prescribe prophylactic antibiotic eye drops. **These drops should be administered as perscribed to the treated eye.**

Precautions & Warnings

- Vision may be temporarily reduced/blurred during treatment period due to presence of CAM360 AG, but should resolve shortly after product removal.
- Should you experience new or worsening symptoms, contact your healthcare provider immediately.