

# CAM360 AmnioGraft™

## Patient Guide

### Use of CAM360 AG with Collagen Shield (CS)



## Up to 72-Hours

### Clinical Application



Mild to Moderate:  
Superficial Punctate Keratitis (SPK)  
Dry Eye Disease (DED)



Neurotrophic Keratitis (NK)  
Stage 1

### Care Instructions

- Ensure hands are cleaned before any contact with your eye during treatment.
- **Apply preservative-free artificial tears at least 3 times a day if eye is not fully patched.**
- It is recommended that the treated eye remain taped for 48- to 72-hours to retain the CS.
- To remove the tape, keep eye shut and gently peel away the tape.
- After tape is removed, continue preservative-free artificial tears at least 3 times a day
- If the CS or tissue falls out, or slips out after tape removal, there is no need for concern. Do not attempt to re-insert the CS or tissue into the eye. Notify your healthcare provider at your follow-up appointment.
- **Avoid** rubbing the eye during treatment as this may dislodge the product or cause discomfort.
- **Avoid** direct water exposure to your eye during treatment.
- **Avoid** ocular exposure to aerosols, hairsprays, cosmetics, lotions, etc.

### Precautions & Warnings

- If you have a known allergy to collagen or bovine products, you must inform your healthcare provider. Do not receive CS treatment. Potential allergic reaction symptoms may include redness, tearing, itching, or swelling.
- You may experience a temporary stinging sensation following the insertion of the CS which should shortly resolve. If not, please advise your healthcare provider.
- Vision may be temporarily reduced/blurred during treatment period due to presence of CAM360 AG, but should resolve shortly after product removal.
- Should you experience new or worsening symptoms, contact your healthcare provider immediately.